JD EVOLUTION

IT

Impianto dentale JDEVOLUTION

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EN

Dental implant JDEVOLUTION

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Dental implant JDevolution is a JDentalCare® implant system. JDentalCare® implant system is intended for surgical placement in the upper or lower jaw. JDentalCare® implant system is comprised of dental implant fixtures and prosthetic devices. JDentalCare® implant system provides a means for prosthetic attachment in single tooth restorations and partially or fully edentulous spans with multiple single teeth utilizing delayed or immediate loading, or as a terminal or intermediary abutment for fixed or removable bridge/overdentures. Prosthetic devices provide support and retention for screw-retained or cemented restorations in mandible and maxilla. JDentalCare® implant system is intended for immediate function on single tooth and/or multiple tooth applications when good primary stability is achieved, with appropriate occlusal loading, in order to restore chewing function.

Contraindications

Placement of JDevolution dental implants may be precluded by patient's conditions that are contraindications for surgery. Absolute contraindications: allergy to titanium, recent myocardial infarction, valvular prostheses, severe kidney disorders, severe liver disorders, treatment resistant diabetes, recent high-dose radiotherapy, severe chronic alcoholism, drug use, terminal illness and tumors. Relative contraindications: chemotherapy, minor kidney disorders, minor liver disorders, endocrine disorders, psychological and psychic disorders, lack of understanding and motivation, AIDS, HIV positive cases, prolonged use of corticosteroids, disorders of the calcium/phosphorus metabolism, erythropoietic disorders, severe nicotine poisoning. JDevolution implants should not be placed in patients where the remaining jaw bone is too diminished or quality bone is poor to provide an adequate primary stability.

Warnings

It is recommended that clinicians, new as well experienced implant users, are updated on the JDentalCare® dental implants and abutments features and their surgical application. For detailed information about method of use of JDentalCare® dental implants and abutments please consult the surgical manuals. Excessive bone loss or breakage of implant may occur when implant is loaded beyond its functional capability. Physiological and anatomic conditions may negatively affect the performance of dental implants. This should be taken into consideration when placing dental implants in patients with the following conditions:

- poor quality bone
- poor oral hygiene*
- inadequate conditions such as blood disorders or uncontrolled hormonal conditions

* Periodic checks for plaque on the zone of the implants through close-up recall sessions (every 4-6 months) in order to prevent the formation of periimplantitis, are indispensable. The device is single use. Do not re-use and dispose after use in accordance to the regulations. Any re-use can jeopardize or seriously aggravate the health status due to risk of infections to the patient and/or the user and for the loss of functionality and efficiency of the device. Small diameter implants and angled abutment are not recommended for the posterior region. The JDentalCare® implant system has not been tested for safety and compatibility in the MR environment. It has not been tested for heating, migration, or image artifact in the MR environment. The safety of JDentalCare® implant system in the MR environment is unknown.

Precautions

JDevolution implant simplifies the surgical techniques required to place dental implants, but the clinician must attend a specific training beyond the supervision of expert implantologists using this technique. Improper technique can lead implant failure or loss of supporting bone or inadequate primary stability or other adverse effects. Thorough screening of implants candidates should be performed including an evaluation of patient capacity to maintain a correct oral hygiene, the patient motivation, the presence of parafunctional habits. Visual inspection as well as periapical radiographs is essential to determine anatomical landmarks, occlusal conditions, periodontal status and adequacy of bone. It is strongly suggested TAC dentascan for providing a real evaluation of dimensions and quality of bone.

Sterility

All implants are delivered sterile and are for single use only prior to the labeled expiration date. Do not re-sterilize. Do not use implants if the package has been damaged or previously opened. Use of non-sterile device may lead to infection of tissues or infectious diseases. Abutments are delivered non sterile and are for single use only. Sterilize prior to use. Re-usable instruments (eg. Drills, drivers) must be sterilized prior to use. Do not re-sterilize the same instrument for more than 30 times. For steam sterilization of the abutments, instruments and the surgical kit of the JDentalCare® implant system, sterilize at 132°C/270°F, for 4 minutes (Autoclave Type: Pre-Vacuum – Drying time: 20 minutes – Wrap type: packaging with sealed containment device). Please refer to product labels for information related to devices sterilization and re-usability.

Potential adverse events

Potential adverse events associated with the use of dental implants may include:

1. Failure of osseointegration
2. Loss of osseointegration
3. Dehiscence requiring bone grafting
4. Perforation of the maxillary sinus
5. Lesion of inferior alveolar nerve
6. Persistent pain, numbness, paresthesia
7. Inflammation or infection
8. Excessive bone loss requiring intervention
9. Implant breakage

Storage and Handling

Devices should be stored at room temperature. Refer to individual product labels and the surgical manual for special storage or handling conditions.

Caution

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

For additional information on surgical procedures please consult the surgical manual available at www.jdentalcare.com or request latest printed version from a JDentalCare representative.

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