






## JDOCTA DESIGN

JDOcta dental implant is designed for the one-stage surgery, that simplifies the treatment, reducing the chair time and providing the patient with earlier esthetic and function. Working at soft tissue level allows easy accessibility to the implant.

### features

### advantages

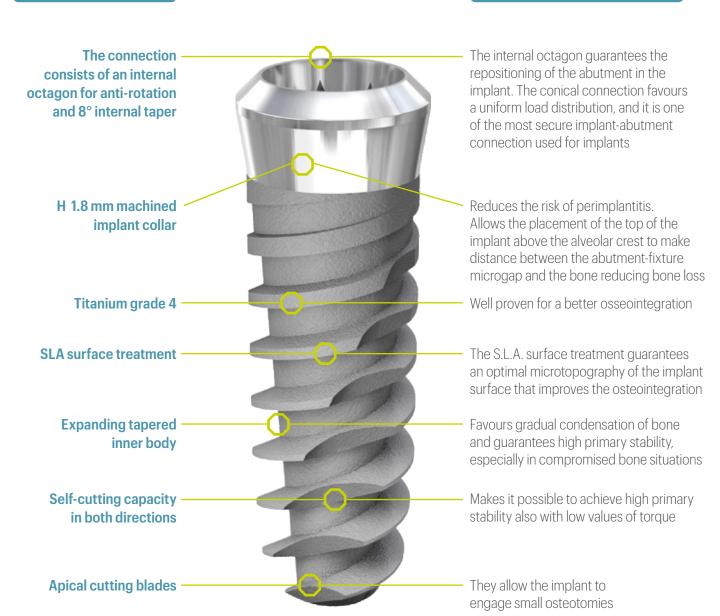




# **PRODUCT SPECIFICATIONS**

The JDOcta dental implants are available in the diameters and lengths shown in the following table:

IMPLANT DIAMETER	TIP DIAMETER	PLATFORM DIAMETER	LENGTH		
Ø 3,7	2,4	4,8	8	10	11,5
Ø 4,3	2,8	4,8	8	10	11,5
Ø 5,0	3,2	6,5	8	10	11,5

Note: all measurements in mm

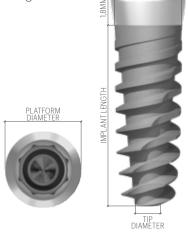


IMPLANT DIAMETER

# **DRILLING PROTOCOLS**

It is recommended to adhere to the indications of the following drilling sequence to ensure optimal primary stability of the implant.

IMPLANT DIAMETER	SOFT BONE TYPE IV	MEDIUM BONE TYPE II-III	DENSE BONE TYPE I
Ø 3,7	2,0 2,4	2,0 2,4 2,8	2,0 2,4 2,8 3,2 (3,6)
Ø 4,3	2,0 2,4 2,8	2,0 2,4 2,8 3,2 (3,6)	2,0 2,4 2,8 3,2 3,6 (4,0)
Ø 5,0	2,0 2,4 2,8 3,2	2,0 2,4 2,8 3,2 3,6 (4,0)	2,0 2,4 2,8 3,2 3,6 4,0 (4,4)

Note: all measurements in mm

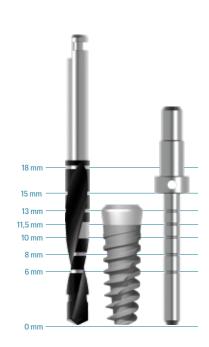

## PRODUCT CATALOGUE

#### Implants:

Ø 3,7

JDOcta RN Ø 3.7 L 8 OC37080 JDOcta RN Ø 3.7 L 10 OC37100 JDOcta RN Ø 3.7 L 11.5 OC37115

Ø 4,3

JDOcta RN Ø 4.3 L 8 OC43080 IDOcta RN Ø 4 3 L 10 OC43100 JDOcta RN Ø 4.3 L 11.5 OC43115

Ø 5,0

JDOcta WN Ø 5.0 L 8 OC50080 JDOcta WN Ø 5.0 L 10 OC50100 JDOcta WN Ø 5.0 L 11.5 OC50115



#### **Cover Screw:**

OCCS Cover Screw RN H 3.0 JDOcta OCCSW Cover Screw WN H 3.0 JDOcta



#### **Prosthetic Screw:**

OCS Prosthetic Screw JDOcta

Torque recommended 30 Ncm



#### Impression Copings Closed Tray / GP Abutments:

Impression Coping Closed Tray / GP Abutment RN JDOcta **OCICCTWC** Impression Coping Closed Tray / GP Abutment WN JDOcta



#### Wax Up Abutments:

OCWAEC Wax-Up Abutment Engaging RN JDOcta **OCWAEWC** Wax-Up Abutment Engaging WN JDOcta

Torque recommended 30 Ncm



#### TiBase:

**OCTIBC** TiBase Engaging RN JDOcta **OCTIBWC** TiBase Engaging WN JDOcta

Torque recommended 30 Ncm



#### **Implant Replicas:**

OCAN Implant Replica RN JDOcta OCANW Implant Replica WN JDOcta



#### **Implant Drivers:**

**OCIDN** Implant Driver JDOcta **OCIDLN** Implant Driver Long JDOcta



#### **Prosthetic Drivers:**

OCSDPF20 Prosthetic Driver for JDTorque L 20 JDOcta OCSDPF25 Prosthetic Driver for JDTorque L 25 JDOcta



JDentalCare, the JDentalCare logo and other trademarks mentioned in this document are, excluding specifically mentioned exceptions, trademarks property of JDentalCare.

The images of the products represented in this brochure are not necessarily to scale.

Note: Not all products manufactured by JDentalCare and represented in this brochure have been regulatory cleared/released for sale in all countries and/or have been licensed in accordance with FDA Rules and Regulations.